

Please hand this sheet to the staff at your CT appointment

CT Protocol: shoulder w/o contrast - (blueprint protocol)

CT Exam Instructions:

Uncompressed, non-encrypted, non-anonymized images, sex, birthdate included.

DICOM Format on CD-

DO NOT USE Auto mA

Using Manual - mA setting of 240 mA will provide compatible images.

Slice thickness must be less than 0.6mm x 0.6mm

Algorithm: Bone, Detector Coverage Minimum

Pitch 0.9 or less

Rotation time 1 sec or less, Exposure Time 1000 ms

Matrix 512x512

DFOV 250mm to 320mm

Image the entire Scapula

The scan should be between 200-450 images,

NO CONTRAST